




ROLLED BANDAGES

A simple bandage rolling “machine” can be created from scrap wood and a wire coat hanger:

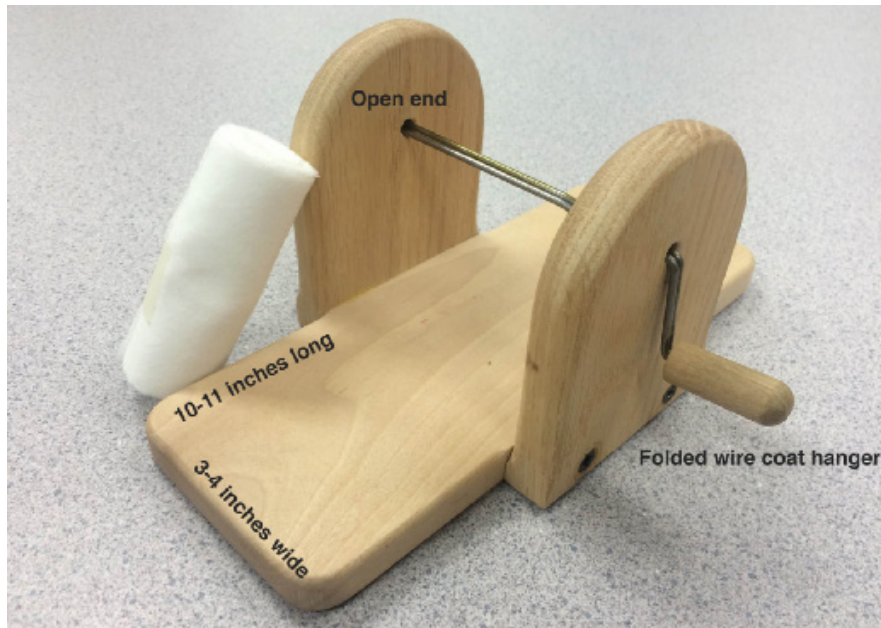

- Pre-wash used **white** sheets. Cut or rip into long strips 2 to 4 inches wide and ideally about 6 feet long (the length of a sheet).
- Place end of bandage strip into the double wire. Roll tightly and evenly.
- Pull the wire from the center of the bandage.
- Note: A couple of backwards turns on the “crank” makes it easy to pull the wire out and avoid an uneven center in the bandage roll.
- Pull or cut off loose threads and secure each bandage with masking tape to keep it from unrolling.



PLEASE MAIL OR DELIVER TO:

GLOBAL HEALTH MINISTRIES
7831 HICKORY STREET NE
MINNEAPOLIS, MN 55432